



## BE-413AG EEG Electrode Set BE-0005 Electrode G (with plastic holder) NE-313AG Earlobe Electrode

### General

- Does not cause oppressive feeling in the patient.
- Easily attached and removed.
- Saline solution is used instead of paste, permitting easy washing of the electrodes.
- Stable measurement because the electrode can be adjusted to fit the scalp surface.
- The electrode contains silver/silver chloride.
- Can be used for both children and adults.
- The pad can be simply attached to the electrode using the O-ring guide.

### Safety Information

**⚠ WARNING** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

**⚠ CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

#### **⚠ WARNING**

Do not connect the electrode lead plug to anything other than the electrode junction box. Failure to follow this warning may cause electrical shock or injury.

#### **⚠ CAUTION**

- Wash the EEG electrode set with cold water or lukewarm water.
- Do not use chlorine bleach, cresol, glutaraldehyde or similar chemicals.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer  
**NIHON KOHDEN CORPORATION**  
1-31-4 Nishiochiai, Shinjuku-ku,  
Tokyo 161-8560, Japan  
Phone +81 3-5996-8041  
<https://www.nihonkohden.com/>



European Representative  
**NIHON KOHDEN EUROPE GmbH**  
Raiffeisenstrasse 10, D-61191 Rosbach, Germany  
Phone +49 6003-827-0 Fax +49 6003-827-599