

# Fastclip Ø3 Child NC-153E

# Fastclip Ø4 Child NC-154E

## General

The NC-153E Fastclip Ø3 and NC-154E Fastclip Ø4 are limb lead electrodes which can be quickly clipped onto a wrist or ankle.

## Safety Information

**⚠ WARNING** A warning alerts the user to the possible injury or death associated with the use or misuse of the instrument.

**⚠ CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

### ⚠ WARNING

When performing MRI test, remove all electrodes from the patient. Failure to follow this warning may cause skin burn on the patient. For details, refer to the MRI manual.

### ⚠ CAUTION

Do not immerse the electrode in water or saline solution for more than an hour. The electrode may get corroded or rusted and the ECG may be distorted.

### ⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer  
**NIHON KOHDEN CORPORATION**  
1-31-4 Nishiochiai, Shinjuku-ku,  
Tokyo 161-8560, Japan  
Phone +81 3-5996-8041  
<https://www.nihonkohden.com/>

**EC REP** European Representative  
**NIHON KOHDEN EUROPE GmbH**  
Raiffeisenstrasse 10, D-61191 Rosbach, Germany  
Phone +49 6003-827-0 Fax +49 6003-827-599