

Surface Stimulation Electrode

NM-420S

General

The NM-420S surface stimulation electrode electrically stimulate the skin surface in nerve conduction study and somatosensory evoked potential measurement.

Safety Information

WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

WARNING

When performing MRI test, remove all electrodes from the patient. Failure to follow this warning may cause skin burn on the patient. For details, refer to the MRI manual.

WARNING

Connect the surface stimulation electrode only to the specified instrument or electrode junction box. Failure to follow this warning may result in electrical shock to the patient.

WARNING

When electrical stimulation is performed for a long period, the felt pads get dry and electrical stimulation may cause skin burn because the dried felt pads increase the skin-electrode contact impedance. Therefore, periodically check that the felt pads are sufficiently moistened with water.

CAUTION

Do not attach the electrode on a wound or inflamed skin.

CAUTION

The area to be stimulated differs according to the nerve or muscle to be examined. Attach the electrode to the appropriate measurement site.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 14 May 2019
2nd Edition: 12 Dec 2022

