

Battery Pack

SB-920P

General

The SB-920P battery pack is a rechargeable battery for a CSM-1000 series bedside monitor.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

Do not do the following to the battery. It may cause leakage, overheating, explosion and fire.

- Short-circuit the + and – terminals on the battery.
- Put the battery into fire or heat the battery.
- Disassemble or modify the battery.
- Give strong impact to or deform the battery.
- Use the battery on unspecified instruments.
- Charge the battery on unspecified instruments.
- Install the battery with the wrong polarity
- Immerse the battery in liquid or get it wet.

⚠ WARNING

If the liquid inside the battery contacts the eyes, mouth or skin, do not rub the eyes, mouth or skin. Wash with clean water and see a physician immediately.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
Raiffisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599