



Sampling Line (Adult, Neonate)

YG-310P, YG-311P

General

The YG-310P sampling line (for adult) and YG-311P sampling line (for neonate) are used with a GF-300 series multigas unit.

Each sampling line connects the water trap on the multigas unit to the respiratory circuit and samples the gas inside the respiratory circuit.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

Do not connect the sampling line to an infusion line, blood pressure measuring line or any line other than the T-piece and flow adapter in the patient vicinity. If the sampling line is not connected properly, the patient cannot be treated and gas cannot be monitored.

⚠ CAUTION

The sampling lines and T-piece are non-sterilized and disposable. Use only for a single patient and single use. Failure to follow this instruction may cause cross infection.

⚠ CAUTION

Connect the T-piece or flow adapter to the respiration circuit and sampling line properly. Otherwise, gas cannot be monitored properly.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

NIHON KOHDEN CORPORATION

1-31-4 Nishiochiai, Shinjuku-ku,

Tokyo 161-8560, Japan

Phone +81 3-5996-8041

<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH

Raiffisenstrasse 10, D-61191 Rosbach, Germany

Phone +49 6003-827-0 Fax +49 6003-827-599