

Capillary Adapter Set

YZ-008B6

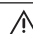

General

For in vitro diagnostic use only.

The YZ-008B6 capillary adapter set is used with a capillary pipette. For applicable capillary pipettes, see “Recommended Capillary Pipette” in the Operator’s Manual.

The capillary adapter set is for aspirating a 20 µL blood sample for measurement on a Nihon Kohden analyzer with the capillary mode.

Safety Information

-  **WARNING** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.
-  **CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator’s Manual.

 **WARNING**

Always wear rubber gloves to protect yourself from infection.

 **WARNING**

During and after setting a glass capillary pipette to the capillary adapter, handle the glass capillary pipette carefully to avoid breaking it.

 **WARNING**

Dispose of the capillary adapter set according to your local laws and your facility’s guidelines (including incineration, melt treatment, sterilization, disinfection and request for waste disposal) for disposing of infectious medical waste. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, it may cause infection.

 **CAUTION**

The capillary adapter is disposable. Do not reuse it. If it is reused, the measurement value may be inaccurate.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator’s Manual or Installation Guide. Therefore, the contents of your Operator’s Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator’s Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiuchi, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

 European Representative
NIHON KOHDEN EUROPE GmbH
Raiffisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599