

Elefix (Paste for EEG)

Z-401CE, Z-181BE, Z-181JE

General

The Elefix EEG paste is designed for attaching Nihon Kohden electrodes in EEG measuring. Refer to the manual for the equipment you use with this paste. Follow the physician's instructions to use the paste.

Safety Information

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ CAUTION

- Use the paste only for EEG measuring.
- If the paste contacts the eyes or mouth, wash thoroughly and immediately with water and see a physician.
- If rash, redness, itch, swelling or abnormality appears on skin, wash the paste off immediately and do not use it.
- Do not leave the paste near the patient or in reach of children.
- If the paste is swallowed, see a physician.

⚠ CAUTION

- Do not use steel wool or any other abrasive to wash the electrodes.
- Do not allow any paste to remain on electrode lead tips because this may cause the tips to rust.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599