

# Software Kit

## QS-251P

### General

The QS-251P software kit is for upgrading the software version of the BSM-2500 series bedside monitor. Installation must be done by a Nihon Kohden representative or a person with technical knowledge. When upgrading the software, contact your Nihon Kohden representative.

- NOTE**
- If more than one BSM-2500 series bedside monitor is used in the same facility, make sure that the bedside monitors have the same software version. If BSM-2500 series bedside monitors with different software versions are used together, correct system operation cannot be guaranteed.
  - Upgrading a bedside monitor deletes all review data on that monitor.

### Safety Information

- ⚠ DANGER:** A danger alerts the user to a hazardous situation which causes death or serious injury.
- ⚠ WARNING:** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.
- ⚠ CAUTION:** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

#### **⚠ WARNING**

Do not leave the software USB memory near the patient or in reach of children. Otherwise it may be swallowed and cause injury.

#### **⚠ CAUTION**

- When inserting or removing the USB memory, turn the power off.
- Do not get the USB memory wet.
- Do not bend or give a strong impact to the USB memory.
- Avoid exposing the USB memory to high temperature or high humidity.

#### **⚠ CAUTION**

The USB memory may be damaged by static electricity. Before handling the USB memory, discharge static electricity from yourself by touching the equipotential grounding terminal on the bedside monitor.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

SHANGHAI KOHDEN MEDICAL  
ELECTRONIC INSTRUMENT CORP.  
No.567 Huancheng Bei Road, Shanghai Comprehensive  
Industrial Development Zone, Fengxian District,  
Shanghai 201401, China  
Phone +86 21-5743-6998  
<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH  
Raiffeisenstrasse 10, 61191 Rosbach, Germany  
Phone +49 6003-827-0 Fax +49 6003-827-599

